



Patent pending.


Liko's Quick-release system is the key to Multirall's flexibility. Multirall is just as appropriate for occasional lifting needs as it is for long-term needs. As a simple, portable overhead lift unit that does not compromise on safety or function, Multirall is unbeatable.

The room-to-room function described on next page facilitates room-to-room transfers with only one lift unit.

Related products



- 3136251 Room to room strap 117
- 3136252 Room to room strap 177
- 3136253 Room to room strap 237

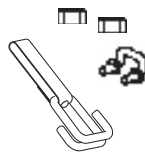

- 3136250 Strap Gripper



- 3136270 Quick-release Hook bolt







- 3136275 Bracket, Quick-release Hook bolt

Start with bringing the patient as close as possible to the doorway.

Connect the Strap Gripper to the Room to Room Strap and lower the lift. The weight of the patient is now gradually transferred to the Room to Room Strap.

When the patient is completely transferred to the Room to Room Strap, the lift strap is disconnected from the Quick-release Hook in room 1.

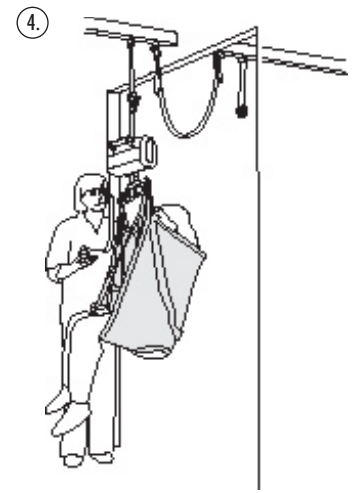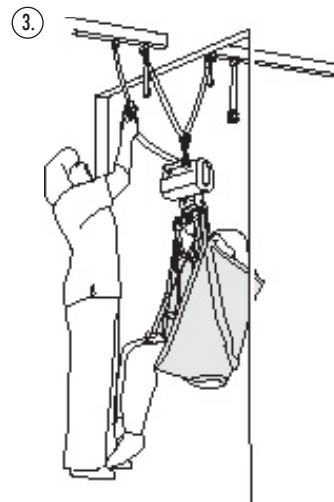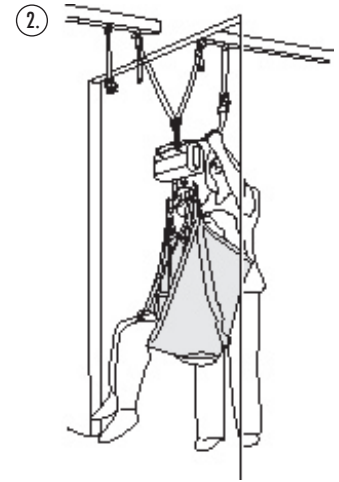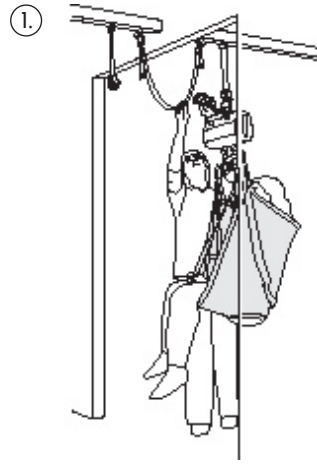

Then connect the lift strap to the Quick-release Hook in room 2 and run up the lift strap.

The patient is now being transferred into room 2.

When the patient has been completely transferred into room 2, the Strap Gripper is released from the Room to Room Strap.

Technical data

Lifting capacity:	200 kg (440 lbs./ 31.4 st)
Lifting interval:	160 cm (63 inch)
Batteries:	2 x 12 V 2.6 Ah, valve regulated, sealed, lead accumulator (gel-type batteries)
Weight:	8.9 kg (19.5 lbs.) incl. batteries
Contact Liko for more information	



Made in Sweden.

Liko is quality certified according to ISO 9001 and environment certified according to ISO 14001.



Liko

www.liko.com

Manufactured by:

Liko AB • Nedre vägen 100
SE-975 92 Luleå, Sweden
info@liko.se

Liko Distributor in UK:

Liko (UK) Ltd • Brunel Centre
Stroudwater Business Park, Stonehouse,
Gloucester, GL10 3RU
Tel: +44 (0)1453 827272
Fax: +44 (0)1453 828844
info@liko.co.uk

National Distributor in USA and Canada:

Liko Inc • 842 Upper Union Street, Suite 4
Franklin, MA 02038, USA
Tel: 508-553-3993, 1-888-545 6671
Fax: 508-528-6642
info@likoinc.com