

Up and over in bed

Everyone knows that it's virtually impossible to lie still in bed – even when we are fast asleep. Even a severely physically disabled person sometimes ends up in the wrong position. Therefore, lifting and repositioning patients is one of the most common bedside procedures in care situations. We also know that it causes many occupational injuries. Liko's positioning sheet, the Repo Sheet Original, is an excellent aid when a patient has to be moved higher up or turned in bed. The sheet is bedded under the patient and can function as a normal bed sheet during the entire care period. Caregivers are then able, with the help of an overhead lift, to simply lift and turn the patient with minimal effort.

The positioning sheet can be used both in combination with, and instead of, an ordinary bed sheet. Furthermore, it is fully possible for one person to move up and turn a patient in bed with the RepoSheet Original.



stat™ RepoSheet Original

Higher up in bed



Whoops! The patient has slipped too far down in the bed. It's high time to move him back up with an RepoSheet Original. The overhead lift is moved into position and the strap loops are hooked to the slingbar.



When the patient has been lifted just clear of the bed surface, he is simply moved higher up with the help of the RepoSheet Original. All with minimal physical effort.



Liko has a full range of overhead lifts. Multirall is a good alternative in combination with an RepoSheet Original.



Bedding is easy with an RepoSheet Original. It is comfortable to lie upon and can be used both in combination with, or instead of, an ordinary bed sheet.

before



Manually lifting a patient higher up in bed was heavy, strenuous work. This procedure is associated with risk and often repeated many times a day for each patient.

Turning in bed



First, lift the patient to one side of the bed with the help of the RepoSheet Original. Then, change the strap loops.



Turn the patient carefully and gently to one side with the help of the overhead lift.



Support the patient with pillows in the appropriate places, unhook the Repo Sheet Original and then straighten and tuck in the sheet.

before



Many patients must be turned several times a day, e.g., to prevent bedsores. Even using correct manual technique, caregivers are exposed to heavy strain. With manual lifting, this was unavoidable.

RepoSheet Original saves backs



Understand, develop and adapt

Lifting a person demands both good assistive devices and insight into medical complications. Liko specializes in the lifting and transfer of physically disabled persons. For more than 25 years, we have successfully developed and manufactured functional assistive devices that are safe and easy to use. Much of our success is based on our unique combination of high-tech expertise, medical knowledge and our own experience of work in the care sector. The Stat RepoSheet Original is a good example of Liko's ability to understand, develop and adapt products to meet different needs in care settings.

stat™ RepoSheet Original

Article	Size	Prod. No.	Max. load
Stat RepoSheet Original	Regular	3687701	440 lbs. (200 kg)
Stat RepoSheet Original	Ultra	3687702	1014 lbs. (460 kg)

Material: Fabric: 50% Cotton, 50% PES.
 Straps: 100% PES.
 Laundering: 85°C. Dry in cabinet dryer or tumble-dry.
 Contact your Liko representative for advice and information about updates.



Headquartered in Sweden, Liko is the global leader in product design and engineering excellence. Liko dealers and representatives are located in more than 25 countries worldwide.

Liko is certified according to the ISO 9001 and ISO 14001 quality and environmental standards systems. Liko's lifts comply with the requirements of ISO 10535.



Lifts with care



www.liko.com

Manufacturer:

Liko AB • Nedre vägen 100
 SE-975 92 Luleå, Sweden
 Tel: +46 (0) 920 - 24 47 00
 Fax: +46 (0) 920 - 24 47 01
 info@liko.se

National Distributor in USA and Canada:

Liko Inc
 842 Upper Union Street
 Franklin, Ma 02038, USA
 Tel: 508-553-3993, 1-888-545 6671
 Fax: 508 -528-6642
 info@likoinc.com

Liko Distributor in UK:

Liko (UK) Ltd • Brunel Centre
 Stroudwater Business Park, Stonehouse,
 Gloucester, GL10 3RU
 Tel: +44 (0)1453 827272
 Fax: +44 (0)1453 828844
 info@liko.co.uk

stat™ | RepoSheet Original



Doc.No: 7U51 6031 0-01 • © Liko AB 2004-10

Lifts with care

